





1. Contract

Fill in all forms, with a pen. Testnumber/contract number from the contract Form is to be noted in own documentation.

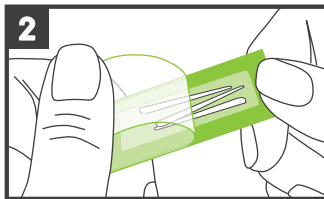


2. Preparation

Take the provided paper points and choose dental pockets for sample taking.

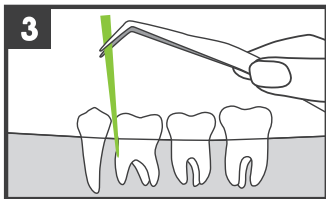

3. Sample taking

Insert each paper point with sterile tweezers into respective dental pocket to the sulcal ground for 10-15 seconds.

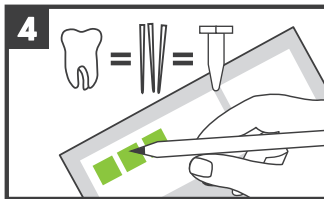

4. Conclusion

Fill in sample taking spots and pocket depths for each tube on the contract form.

2a. Poolsample

Samples from more than one dental pocket are analysed at once. Put all used paper points in one tube.

2b. Single determination

Separate Analysis of up to 4 dental pockets. One paper point per dental pocket and tube respectively.

Pre sample taking: Removal of supra-gingival plaque und drying of respective pocket (no professional tooth cleaning).




5. Shipping

Send samples and form inside the iai Testkitbox free of charge (no envelope needed) and receive the result online (with registration) and via mail (if needed).




1. Commande

Remplir lisiblement le bon de commande avec un stylo. Noter le numéro du test dans vos documents.

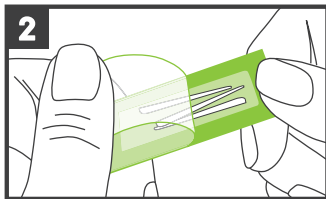


2. Préparation

Préparer les pointes de papier et déterminer les points de prélèvement.

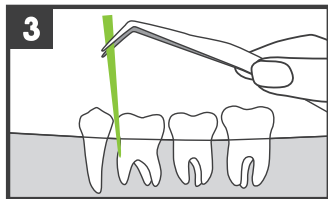

3. Prélèvement

Introduire la pointe de papier avec une pincette stérile dans la poche gingivale jusqu' au sol. Laisser la pointe pendant 10-15 secondes.

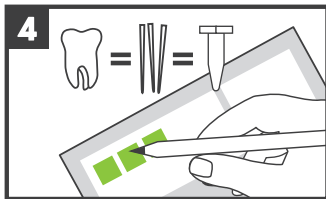

4. Résumé

Noter les points de prélèvement et la profondeur des poches pour chaque tube sur le bon de commande.


2a. Pool

Echantillons de plusieurs poches gingivales sont analysés ensemble. Placer toutes les pointes de papier dans un seul tube.

2b. Single-site test

Analyse particulière d'une jusqu'à 4 poches gingivales différentes. Chaque pointe de papier pour tube et poche gingivale.

Avant le prélèvement: Eliminer la plaque supra-gingivale et sécher le point de prélèvement (pas de nettoyage dentaire professionnel)

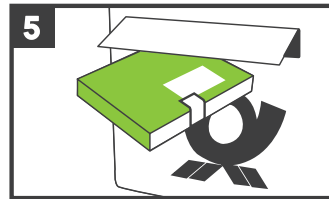

5. Expédition

Expédition gratuite des documents et des tubes utilisés dans la iai test-boîte. Recevoir le résultat en ligne (après enregistrement) ou par courrier (si souhaité).